

# Carnation Verruca & Wart Remover Freeze Spray



**Category**  
Footcare  
**Manufacturer**  
CUXSON GERRARD & CO LTD  
**Pack size**  
50ml  
**RRP**  
£11.99  
**Legal Status**  
MD



## Uses

Verruca and wart removal device



## Format and Ingredients

Freeze spray containing:  
Dimethyl Ether, Butane, Isobutane, Propane,



## Dosage

Adults and children over 4 years: one 30 second application.  
To use: attach an applicator firmly into the spray nozzle on the can. Place the protective cap over the applicator and spray head ensuring both 'Y' shapes fit together. With the can upright press down on the protective cap for up to 3 seconds to saturate the applicator. Remove the protective cap and keep for future treatments. Lightly place the tip of the applicator on the verruca or wart for a maximum of 30 seconds - the verruca or wart will usually disappear or fall off over the next 10-14 days. Never push the protective cap while it is in contact with or above the skin. Each can comes with 12 disposable applicators.